22655

ProPig

- DE Bedienungsanweisung
- FR Mode d'emploi
- EN Instructions for use
- IT Istruzioni per l'uso
- ES Instrucciones de uso
- PT Instruções de uso
- NL Gebruiksaanwijzing
- SE Bruksanvisning
- FI Käyttöohje
- DK Brugsanvisning
- NO Bruksanvisning
- PL Instrukcja obsługi
- HU Használati útmutató
- SK Návod na použitie
- CZ Návod k použití
- SI Navodila za uporabo
- HR Naputak za uporabu

KERBL

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Safety and operating instructions

1. Safety instructions

It is only possible to work safely with this device by reading the corresponding operating and safety instructions in their entirety and following them to the letter.

Application

The warranty issued by the manufacturer and any liability claims against the manufacturer shall be rendered null and void if the device is used in a manner not in accordance with its designated use or if alterations are made to it. Check that the voltage given on the nameplate is the same as your source at the mains.

Check all components before use

Have any damaged parts repaired by a specialist or the manufacturer. Incorrectly performed repairs represent a risk of accidents to the user. Always use original spare parts for any repairs.

Restrict unauthorised access

Make sure that no-one, in particular children, can get near the tooth grinder without your permission.

Keep your workplace tidy

An untidy workplace increases the danger of accident.

Protect connection cables

Do not use the connection cable to pull out the plug or as a means of carrying the device. Make sure that connection cables are not exposed to heat and do not come into contact with oil or sharp edges. Damaged connection cables represent a potential cause of fire, short-circuits and electric shocks.

Take the ambient conditions into account

Protect your device against all liquids and moisture. Otherwise there will be an increased risk of fire or electric shocks.

Look after your device

Always keep your product in a safe, dry place out of the reach of children. Pay attention to any maintenance requirements. Check your device at regular intervals. Always use original accessories and spare parts.

National and international regulations

Comply with national and international regulations governing health and safety at work.

2. Instructions for use - Tooth Grinding Machine for piglets 12-18 Volt

Why grind teeth?

The teeth of sucking pigs are sharp and pointed and can easily damage the sow's udder and teats. To protect the sow's udder and also to ensure calm and harmonious suckling it used to be customary to cut the teeth of sucking pigs. This also prevented bite injuries to the rest of the litter. Tooth cutting is a painful procedure for a sucking pig. When they are cut with a pair of tooth-nippers, even if the tooth is cut correctly, vertical cracks can easily occur in the dental enamel and the pulp of the tooth (nerves, blood vessels, connective tissue) becomes exposed. The cracks constitute a very high risk of future infection for the pigs. In many cases these infections are not discovered until slaughter.

The alternative is tooth-grinding!

The tooth-grinding machine makes work easier, reduces considerably the risk of infection and is kinder to the piglet. Just a little pressure from the grinding machine removes the tip and sharp edges of the tooth. This protects the sow from being bitten and gives the little pigs a good start.

Assembly/Disassembly of the Grinding Body

Pull the mains plug out before changing tools. Never press the locking pin while the machine is running. 1. Turn the spigot nut gently while pressing the locking pin until it engages. 2. Open the clamping jaw and change the tool. Clamp all tools as short as possible. Shafts, which protrude too far will easily bend causing the machine to vibrate. As next, screw on the protective aluminium cap. The grinding body must not stand out over the protective aluminium cap. If this happens, the grinding body has to be loosened again, the pen inserted deeper into the device and locked again. Do not work without wearing protective goggles. Do not work with damaged grinding discs or deformed shafts. If the unit overheats, switch it off and allow it to cool for a few minutes.

Care

The grinding body should be always cleaned after use Grinding body and protective aluminium cap should be cleaned with a stiff brush and water or vinegar. It is also possible to use a conventional lime cleaner for a basic clean-up of both parts. After cleaning lubricate the attachment of the grinding body with one drop of oil.

Helpful Hint: Do not push the grinding pen against the teeth. This reduces the grinding effect and the grinding body will be worn unnecessarily. Only grind the teeth at the top so that only the sharp edges will disappear.

3. TECHNICAL DATA

Revolutions of 20,000 pm for ca. 40 Watt

Wearing Parts

Protective Aluminium Cap

Ref. No. 22878

Grinding Body

Ref. No. 22913

Charging the storage battery pack:

- The tooth grinder is supplied with electricity via a separate battery pack.
- This should be charged for approximately 3 hours before initial use.
- Remove the storage battery from the device when charging.
- Plug the transformer's appliance connector plug into the charging socket (IN) on the storage battery pack.
- Plug the transformer into the socket. The charging control display (I) on the storage battery pack should light up.
- When charging is complete, the charging control display (I) on the storage battery pack will begin to flash.
- Recommendation: it is possible to extend the life of the storage battery by only recharging it after experiencing a noticeable loss of grinding power. Full electrical capacity is reached after charging for a period of approximately 3 hours.
- The tooth grinder can be used for over 2.5 hours when the storage battery pack is fully charged. This capacity is reached after approximately 5 charging cycles.

Operation:

- Only connect device to the storage battery pack when switched off!
- Fasten support belt. You can now comfortably attach the storage battery pack to the belt using the attachment clip. The tooth grinder can then be placed in the specially designed bag.
- Plug the tooth grinder's connecting plug into the socket (OUT) on the storage battery pack.
- Switch on the device using the on/off switch.
- Switch the device off after use.
- Always switch off the device during breaks taken in the grinding process!

Electronic scrap

Disposing of this device after its service life is the responsibility of the operator. Please consult the valid national regulations. The device must not be disposed of in household waste. In accordance with the stipulations of the EU Directive on the Disposal of Electrical and Electronic Devices, the device can be disposed of free of charge at the local waste collection or recycling centre. Alternatively, it can be returned to retailers who offer a collection service. The proper disposal helps to ensure environmental protection and prevents any adverse effects on human health and the environment.

CE-symbol/CE-conformity declaration

Albert KERBL GmbH herewith declares that the product/machinery described in these instructions conforms with the basic requirements and the other relevant conditions of regulations 2004/108/EG, 2006/42/EG and 2006/95/EG. The CE-symbol stands for fulfilment of the European Union guidelines. The conformity declaration can be viewed at the following Internet address: <http://www.kerbl.de>

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EN WARRANTY CARD

In accordance with our general terms and conditions, we are issuing a 2 year warranty in order to remediate damages caused by material or manufacturing flaws for this device. The warranty begins at purchase date.

The warranty does not cover:

- 1) Damages caused by normal wear (wear parts, anti-slip parts), capacity overload, and improper usage (non-observance of the instruction manual).
- 2) Damages caused by force majeure or by transport.
- 3) Damage of rubber parts such as those found in connecting lines.

Warranty repairs will only be done directly at the manufacturing plant or at authorized, contracted repair shops! The manufacturer reserves the right to reject every warranty claim if the warranty has been breached. For this purpose, the assembled, packed and post-free device must be sent or brought back to the point of sale together with the original receipt and the warranty card. The risk is carried by the sender. (Address please see retailer stamp). Please describe the complaint or concern as precisely as possible. Costs that arise due to unjustified claims are to be paid by the owner of the device.

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Albert Kerbl GmbH
Kerbl Austria Handels GmbH