TYPE: 22991

PREMIUM

(DE) Zahnschleifgerät

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Safety and Operating Instructions

I. Safety instructions

It is only possible to work safely with this device by reading the corresponding operating and safety instructions in their entirety and following them to the letter.

1. Application

The warranty issued by the manufacturer and any liability claims against the manufacturer shall be rendered null and void if the device is used in a manner not in accordance with its designated use or if alterations are made to it.

Check that the voltage given on the nameplate is the same as your source at the mains.

2. Check all components before use.

Have any damaged parts repaired by a specialist or the manufacturer. Incorrectly performed repairs represent a risk of accidents to the user. Always use original spare parts for any repairs.

3. Restrict unauthorised access.

Make sure that no-one, in particular children, can get near the tooth grinder without your permission.

4. Keep your workplace tidy.

An untidy workplace increases the danger of accident.

5. Protect connection cables.

Do not use the connection cable to pull out the plug or as a means of carrying the device. Make sure that connection cables are not exposed to heat and do not come into contact with oil or sharp edges. Damaged connection cables represent a potential cause of fire, short-circuits and electric shocks.

6. Take the ambient conditions into account.

Protect your device against all liquids and moisture. Otherwise there will be an increased risk of fire or electric shocks.

7. Look after your device.

Always keep your product in a safe, dry place out of the reach of children. Pay attention to any maintenance requirements. Check your device at regular intervals. Always use original accessories and spare parts.

8. National and international regulations

Comply with national and international regulations governing health and safety at work.

II. Instructions for use – Tooth Grinding Machine for piglets 230 Volt

Why grind teeth?

The teeth of sucking pigs are sharp and pointed and can easily damage the sow's udder and teats. To protect the sow's udder and also to ensure calm and harmonious suckling it used to be customary to cut the teeth of sucking pigs. This also prevented bite injuries to the rest of the litter.

Tooth cutting is a painful procedure for a sucking pig. When they are cut with a pair of tooth-nippers, even if the tooth is cut correctly, vertical cracks can easily occur in the dental enamel and the pulp of the tooth (nerves, blood vessels, connective tissue) becomes exposed. The cracks constitute a very high risk of future infection for the pigs. In many cases these infections are not discovered until slaughter.

The alternative is tooth-grinding!

The tooth-grinding machine makes work easier, reduces considerably the risk of infection and is kinder to the piglet. Just a little pressure from the grinding machine removes the tip and sharp edges of the tooth. This protects the sow from being bitten and gives the little pigs a good start.

Assembly/Disassembly of the Grinding Body

A sleeve nut attaches the grinding pen at the device. For locking the grinding body you have to hold down the button situated directly at the handle, while you fix it using the key enclosed. As next, screw on the protective aluminium cap. The grinding body must not stand out over the protective aluminium cap. If this happens, the grinding body has to be loosened again, the pen inserted deeper into the device and locked again.

Care

The grinding body should be always cleaned after use. Grinding body and protective aluminium cap should be cleaned with a stiff brush and water or vinegar. It is also possible to use a conventional lime cleaner for a basic clean-up of both parts. After cleaning lubricate the attachment of the grinding body with one drop of oil.

Helpful Hint

Do not push the grinding pen against the teeth. This reduces the grinding effect and the grinding body will be worn unnecessarily.

Only grind the teeth at the top so that only the sharp edges will disappear.

III. Technical Data

Revolutions of 5,000 to 20,000 pm for variable adjustment, 100 watt, 50 hz

Wearing Parts:

Protective Aluminium Cap	Ref. No. 22966
Grinding Body	Ref. No. 22914

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WARRANTY CARD

According to our General Conditions of Shipment for Products or Services
we concede on the above appliance

1 YEAR GUARANTEE

Defect liability starts from the date of purchase.

This guarantee covers non of the following:

- 1) Damages caused by non-observance of the operation instructions.
- 2) Damages caused by force majeure as well as by transport.
- 3) Damaged rubber parts as e.g. power cords etc. are excluded from warranty.

Repairs during the warranty period will only be carried out by the manufacturer!

For this purpose the appliance has to be packaged properly and forwarded postage free at customer's risk together with the original invoice and the warranty card to the dealer (see dealer stamp).

Please give detailed information on the kind of defect.

Costs resulting from unjustified customer complaints are at the expenses of the customer.

Date: _____

Your dealer: _____

Device number: _____

84428 Buchbach · Felizenzell 9 · Tel +49(0) 80 86/9 33-100 · Fax +49(0) 80 86/9 33-500